



≡ UNCINT



U&C International Co., Ltd.

Seoul Office

Room 408, 28, Digital-ro 30-gil, Guro-gu, Seoul, Korea

Head Office

106-11 Beomjigi-ro, Danwon-gu, Ansan-si, Korea

Tel. +82 2 864 7715 Fax. +82 2 864 7713

Contact. marketing@uncint.net

www.uncint.com

ABOUT US

UNC International is a dental zirconia manufacturer founded in 2013.

We work hard with professional knowledge and sales skills with the aim of manufacturing the world's best zirconia. As a result, we have grown to more than 60% market share in Korea.

Now we're heading to the global market with our technology and vision.

OUR VALUES

People Centered

At UNC we understand the importance of not just health but also beauty. We invent products not just for those in need of medical reconstruction, but also those looking to upgrade their smiles.

Dedicated to Our Service

Our dedicated team members are ready to answer any questions or concerns via phone or email, and handle any issues whether it be domestic or abroad.

Innovating the Future Market

We are focused on creating new, never seen before Zirconia products. We hope to become the future of dental health and beauty and are making steps towards that.



OUR HISTORY

2013

- Founded the UNC (KICET)
- Registered Venture Company
- Licensed manufacturing medical instrument
- Obtained KGMP Certification
- Established U&C Int. R&D center (Korea Industrial Technology Association)
- Launched dental Zirconia block I-JAM

2014

- Launched I-JAM Sapphire, Ruby, Opal, New ruby
- Launched ZirMic1, ZirMic2 (First anterior zirconia block in Korea)

2015

- Export contract with Russia and Mexico

2016

- Export and manufacturing contract with China
- Supply contract with 3M Corp
- Changed into a juristic person U&C International
- Launched RAZOR 1100, 1300 (First new material block in Korea)

2017

- Launched Razor 800 for anterior teeth (First anterior high transmittance new material block in Korea)
- Launched RAZOR Coloring Liquid Natural, Effect
- Established domestic and international dental marketing and distribution company U&T
- Achieved NO.1 market share in Korea (40%)

2018

- Relocated new factory in Guro-gu, Seoul
- Launched 3D Multilayer EVEREST AT, PT
- Participated in VIDEK 2018 in Ho Chi Minh, Vietnam

2019

- Approved ISO 13845, 9001
- Approved FDA
- Launched EVEREST line up (AT, PT, MO)
- Established Roydent Lab in San Jose, USA
- Participated in AEEDC 2019 in Dubai, UAE
- Participated in Dental-Expo 2019 in Moscow, Russia
- Participated in HIDEK 2019 in Hanoi, Vietnam

2020

- Obtained certification in Vietnam
- Established overseas sales department
- Established a local sales corporation in Ho Chi Minh, Vietnam (UNC International SEA)
- Relocated new headquarter & factory in Danwon-gu, Ansan-si
- Participated in AEEDC 2020 in Dubai, UAE
- Participated in LMT Lab Day 2020 in Chicago, USA

2021

- Obtained MDSAP Certification (Canada, Australia, Japan, USA)
- Obtained ARTG Certification (Australia)
- Launched EVEREST Volcan (Dental electronic furnace)
- Established STL Digital Art Center (Korea and Ho Chi Minh, Vietnam)
- Participated in AEEDC 2021 in Dubai, UAE
- Participated in IDS 2021 in Koeln, Germany

2022

- Launched EVEREST Multilayer UVS
- Launched EVEREST Volcan Speed (Speed sintering furnace)
- Participated in LMT Lab Day 2022 in Chicago, USA
- Participated in IDEM 2022 in Singapore
- Participated in Dental-Expo 2022 in Moscow, Russia

2023

- UNC International welcomes a new CEO
- Participated in AEEDC 2023 in Dubai, UAE
- Participated in IDS 2023 in Koeln, Germany
- Participated in World Dental Show 2023 in Yokohama, Japan
- Participated in CADEX 2023 in Almaty, Kazakhstan
- Participated in TDA 2023 in Bangkok, Thailand

2024

- Started domestic general materials distribution business
- Revamped EVEREST Multilayer UVS
- Revamped EVEREST Volcan Speed & Mini
- Participated in IDEM 2024 in Singapore
- Participated in VIDEK 2024 & held Seminar in Ho Chi Minh, Vietnam
- Participated in AMIC 2024 in Mexico City, Mexico

2025

- Participated in CIOSP 2025 in Sao Paulo, Brazil
- Participated in AEEDC 2025 in Dubai, UAE
- Participated in IDS 2025 in Koeln, Germany
- Participated in IDEX 2025 in Istanbul, Turkiye
- Participated in VIDEK 2025 in Hanoi, Vietnam
- Participated in CADEX 2025 in Almaty, Kazakhstan

GLOBAL PARTNERS



• SOUTH AMERICA

PERU / BRASIL / CHILE
ECUADOR / COSTA RICA

• NORTH AMERICA

CANADA / UNITED STATES

• EUROPE

ALBANIA / KOSOVO / RUSSIA
NORTH MACEDONIA / BULGARIA
GREECE / HUNGARY / POLAND
MOLDOVA / ITALY / ROMANIA

• AFRICA

MOROCCO / LIBYA / TUNISIA

• ASIA

INDIA / THAILAND / SINGAPORE
JAPAN / HONG KONG / TAIWAN
VIETNAM / MALAYSIA / INDONESIA
TURKIYE / PALESTINE / ISRAEL
IRAQ / IRAN / UAE /
KAZAKHSTAN / UZBEKISTAN

• OCEANIA

AUSTRALIA / NEW ZEALAND

PRODUCT LIST

3D MULTILAYER

EVEREST UVS



EVEREST AT



EVEREST AT+



EVEREST PT



STRENGTH

Incisal 700MPa
Cervical 1150MPa

Incisal 800MPa
Cervical 1100MPa

800MPa

Incisal 1100MPa
Cervical 1250MPa

TRANSLUCENCY

Incisal 49%
Cervical 45%

Incisal 48%
Cervical 46%

48%

Incisal 46%
Cervical 44%

INDICATION

Full Zirconia
Anterior, Posterior
Single Crown to Full Arch

Full Zirconia
Anterior
Veneer, Crown, Bridge

Full Zirconia
Anterior
Veneer, Crown, Bridge

Full Zirconia
Posterior
Crown, Bridge, Long-span
Bridge, All on 4

RAZOR 800



RAZOR 1100



RAZOR 1300



RAZOR CAP



MONOLAYER

800MPa

1100MPa

1300MPa

1400MPa

48%

46%

44%

35%

Full Zirconia
Anterior
Veneer, Inlay, Crown,
Bridge

Full Zirconia, Coping
Anterior, Posterior
Inlay, Coping, Crown,
Full Arch, All on 4

Full Zirconia, Coping
Posterior
Crown, Bridge, Full Arch,
All on 4

Coping, Coping Bridge

EVEREST 3D MULTILAYER UVS




EVEREST Multilayer Universal Speed (UVS) combines our latest multilayer zirconia technology to create a strong but transparent material with no compromise to shade stability in both speed and normal sintering. Suitable for single crowns and full arches, no variation in translucency or strength.


	Translucency	Strength	Block Size
Incisal			Ø 98
	49%	700MPa	12mm
	47%	900MPa	14mm
	45%	1150MPa	16mm
	45%	1150MPa	18mm
			20mm
			22mm
			25mm
			30mm
Cervical			

Sintering Schedule


Fast mode (Only single)
Max. temp: 1500°C
Total hours: About 40min

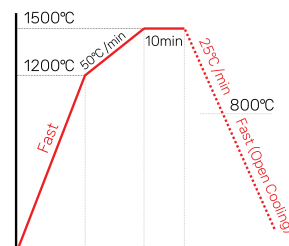

Normal mode (Below 6units)
Max. temp: 1500°C
Total hours: About 4hr

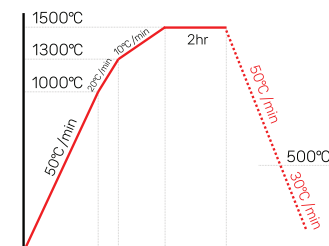

Long term mode (Above 7units)
Max. temp: 1500°C
Total hours: About 11hr

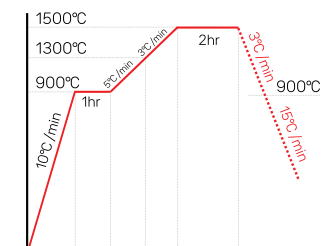


Shades

A1, A2, A3, A3.5
B1, B2, B3
C1, C2, C3
D2, D3
BL1, BL3

14 Shades

Indication

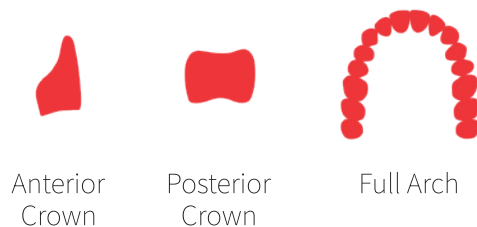


Anterior Crown

Posterior Crown

Full Arch

Why EVEREST Multilayer UVS?

- Fast sintering zirconia
- 40 minute sintering cycle (based on single crown)
- Same results in both Normal and Speed sintering
- Flexural strength, translucency, and shade stability
- For all indications (single crown to full arch, anterior and posterior)

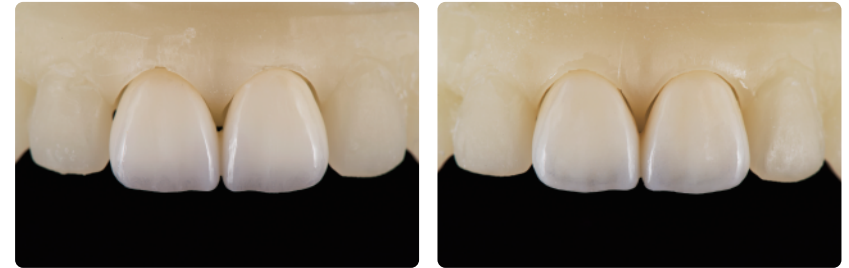

1500°C
Normal Sintering
4hour

=

1500°C
Fast Sintering
40min

EVEREST Volcan Speed & Volcan Speed MINI

Paired with EVEREST Multilayer UVS, sinter a crown in only 40 minutes and full arch in only 4 hours.

- 40 minute speed sintering mode
- Speed sintering- Maximum 5ea crowns
- Shade stability and consistency
- High purity and heat durability



Zirconia Pre-Polishing Felt

UNC Pre-Polishing Felt is infused with diamond paste in order to save time when polishing and provides a fast high gloss effect.

Paired with EVEREST Multilayer UVS and EVEREST Volcan Speed, finish restorations within one visit.

- Designed for polishing zirconia before sintering
- Pre-polishing can save up to 30 minutes
- Under 1 minute for a single crown




EVEREST 3D MULTILAYER AT



EVEREST Multilayer AT is a very high transparency multilayer zirconia designed for aesthetic prosthetics in the anterior region.

	Translucency	Strength
Incisal		
	48%	800MPa
	48%	800MPa
	46%	1100MPa
	46%	1100MPa
Cervical		

Indication

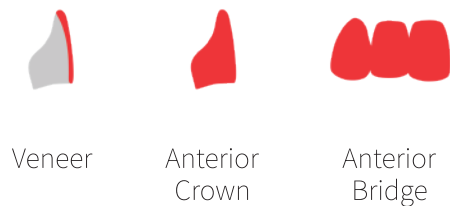


Veneer

Anterior
Crown

Anterior
Bridge

Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1530°C
Total hours: About 8hr

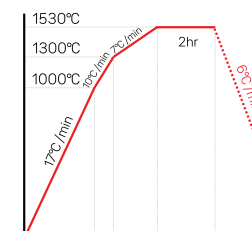

Shades

A1, A2, A3, A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4
BL0, BL1, BL2, BL3

20 Shades


Block Size

Ø 95, 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm
30mm


EVEREST MULTILAYER AT+

EVEREST Multilayer AT+ is a very high transparency multilayer zirconia designed for aesthetic prosthetics in the anterior region.

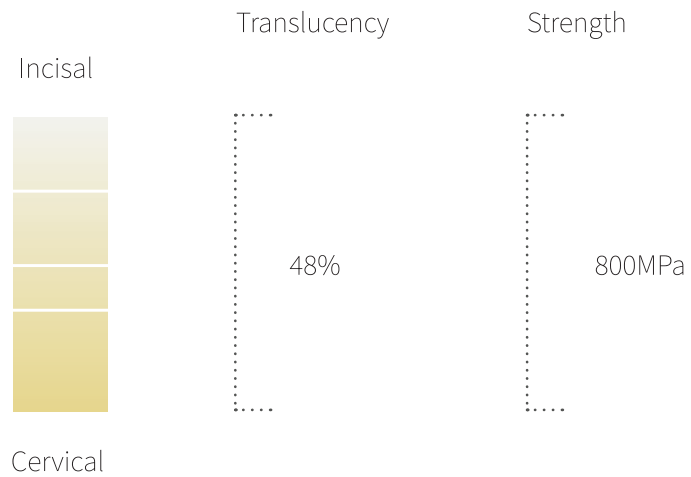

Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1500°C
Total hours: About 4hr

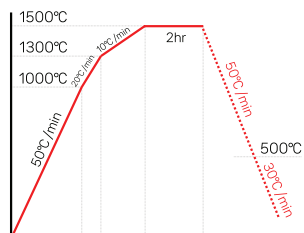

Shades

A1, A2, A3, A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4
BL0, BL1, BL2, BL3

20 Shades

Block Size

Ø 95, 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm
30mm

Indication



Veneer





Anterior
Crown



Anterior
Bridge

EVEREST 3D MULTILAYER PT


EVEREST Multilayer PT is a high-strength aesthetic multilayer zirconia designed for large size prosthetics and posterior crowns that need the strength to resist strong biting pressures.

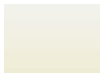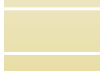	Translucency	Strength
Incisal		
	46%	1100MPa
	46%	1100MPa
	44%	1250MPa
	44%	1250MPa
Cervical		




Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1530°C
Total hours: About 8hr

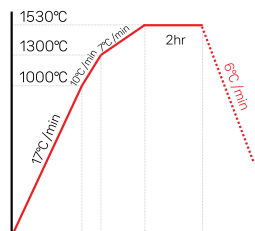

Long term mode (Above 7units)
Max. temp: 1530°C
Total hours: About 11hr 30min

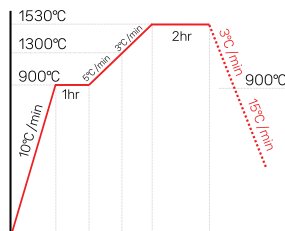

Shades

A1, A2, A3, A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4
BL0, BL1, BL2, BL3


20 Shades

Block Size

Ø 95, 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm
30mm

Indication

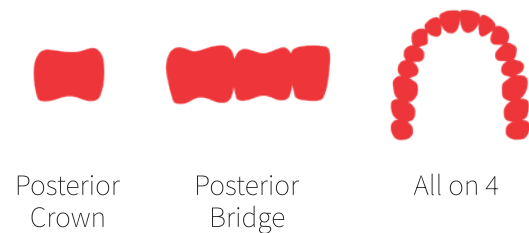

RAZOR 800



Razor 800 is a zirconia disc with a flexural strength of 800 MPa, a very highly transparent single layer zirconia designed for anterior restorations.

Translucency

Strength



48%

800MPa

Indication



Veneer



Inlay




Anterior
Crown



Anterior
Bridge

Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1500°C
Total hours: About 4hr

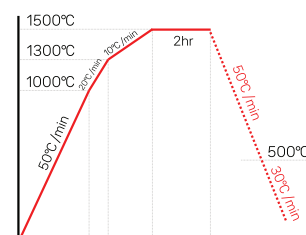

Shades

A0, A1, A2, A3,
A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4

17 Shades


Block Size

Ø 95, 98

10mm	20mm
12mm	22mm
14mm	25mm
16mm	30mm
18mm	


RAZOR 1100

RAZOR 1100 is a versatile zirconia disk that can be used for both anterior and posterior restorations. It allows high quality prosthetics to be produced with a strong flexural strength of 1100 MPa and a high transparency of about 46%.

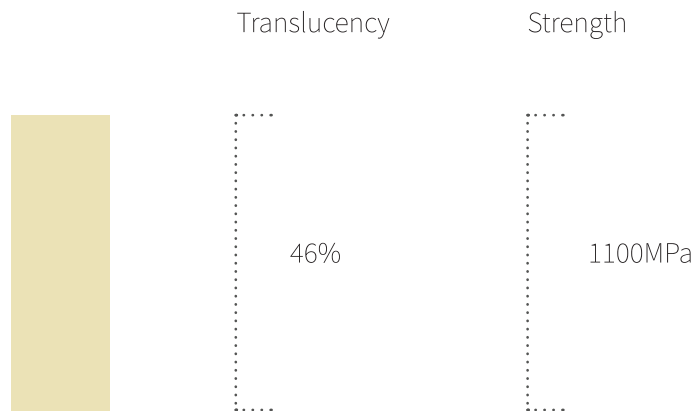


Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1530°C
Total hours: About 8hr

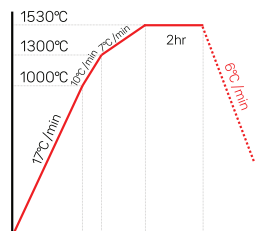

Long term mode (Above 7units)
Max. temp: 1530°C
Total hours: About 11hr 30min

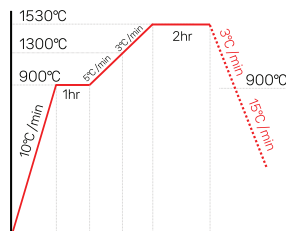

Shades

A0, A1, A2, A3, A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4
BL1, BL3


19 Shades

Block Size

Ø 95, 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm
30mm

Indication

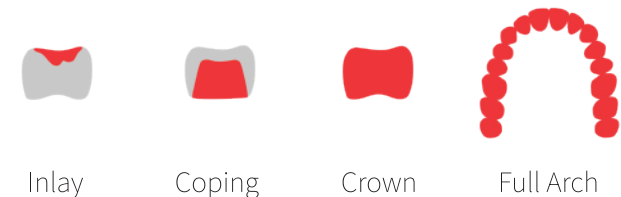

RAZOR 1300

RAZOR 1300 is a high-strength zirconia disc with a very strong flexural strength of 1300 MPa. You can use this to make full-arch cases and copings very safe and beautiful.



Translucency



44%

Strength



1300MPa

Indication



Posterior Crown



Coping




Coping Bridge




Full Arch

Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1530°C
Total hours: About 8hr

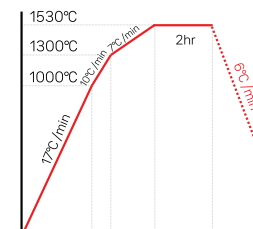

Long term mode (Above 7units)
Max. temp: 1530°C
Total hours: About 11hr 30min

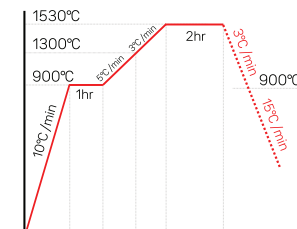

Shades

A0, A1, A2, A3, A3.5, A4
B1, B2, B3, B4
C1, C2, C3, C4
D2, D3, D4

17 Shades


Block Size

Ø 95, 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm
30mm


RAZOR CAP

RAZOR CAP is a dedicated product for zirconia coping production. It has a flexural strength of 1400 MPa, and can be used for single caps to full arch's. This product can also be used in cases where you need to cover inner color, such as discolored teeth or to coat an implant abutment.

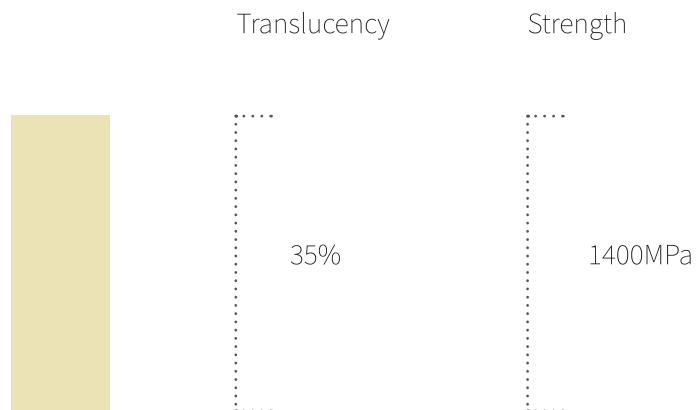


Sintering Schedule

Normal mode (Below 6units)
Max. temp: 1530°C
Total hours: About 8hr

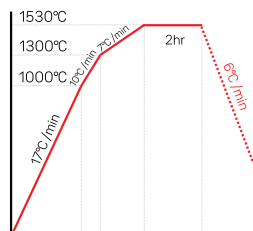

Long term mode (Above 7units)
Max. temp: 1530°C
Total hours: About 11hr 30min

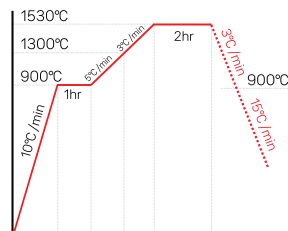

Shades

A0 (WHITE)

Block Size

Ø 98

12mm
14mm
16mm
18mm
20mm
22mm
25mm

Indication



Coping



Coping
Bridge

EVEREST VOLCAN SPEED

EVEREST Volcan Speed was developed and optimized for the production of one-day zirconia prosthetics. Quickly sintered zirconia prosthetics can be produced in 40 minutes from room temperature to sintering completion. It can also be used with a normal sintering function.

Used with UNC's fast-sintering multilayer zirconia block Everest Universal Speed (UVS) to see the best results.

Volcan Speed Specifications

*A two-layer tray is also available

Product Dimension :	300(W) x 400(D) x 640(H)
Chamber Dimension :	Ø100mm x 80mm
Weight :	About 41kg
Max. Temp. :	1600°C (2912°F)
Heater :	MoSi ₂ 1900 Grade
Power :	AC 220V, 50/60 Hz

Volcan Speed MINI Specifications

Product Dimension :	270(W) x 380(D) x 570(H)
Chamber Dimension :	Ø70mm x 60mm
Weight :	About 36kg
Max. Temp. :	1600°C (2912°F)
Heater :	MoSi ₂ 1900 Grade
Power :	AC 220V, 50/60 Hz



ZIR GUIDE

The ZIR GUIDE is a zirconia shade guide created from our EVEREST Multilayer AT zirconia block.

The ZIR GUIDE comes in 18 colors, including the bleach colors BL1 and BL3. By using our own products and real zirconia for our shade guide, we are able to guarantee accurate shades throughout our line, with even easier matching.

The ZIR GUIDE can also be used with other EVEREST zirconia blocks such as PT and UVS, as the colors are made identically.



Patient shade matching



After setting

COLORING LIQUID



EVEREST Coloring Liquid is a universal coloring liquid that can be used on any zirconia disks. It is suitable for various clinical aesthetic cases and creates a beautiful and natural finishing product.*

Effect	Dentin
	Dipping (RAZOR A0 : WHITE) : U&C Natural
Incisal	A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, D4, BL1, BL3 50ml
Blue	
Gray	
Violet	Brushing (Pre-shade : 1100, 1300, 1400, MO, CAP) : U&C
Orange	Natural A1, A2, A3, A3.5, A4 50ml
Brown	
White	
Pink	
Opaque	

* Both dipping and brushing is applicable.
User is responsible for application method and testing.



UNC International



uncinternational_kr



@uncint



www.uncint.com



marketing@uncint.net



TEL: +82-2-864-7715

FAX: +82-2-864-7713

Seoul Office

Room 408, 28, Digital-ro 30-gil, Guro-gu, Seoul, Korea

Head Office

106-11 Beomjigi-ro, Danwon-gu, Ansan-si, Korea

* Product packaging and design is subject to change.