**A. Indication for use**

U&C Liquid can be used as an accessory to zirconium dioxide dental restorative material to provide individual tooth (or teeth) shading. It is intended to be used solely by certified dental technicians for fabrication of zirconium dioxide restorations for individual dental patients.

Caution: Federal law restricts this device to sale by or on the order of dental health professionals.

**B. Product description**

U&C Liquid is water-based solution. It is used for the individual staining of dental zirconia frameworks and restorations prior to the final sintering. It enables trained dental technicians to adjust the restoration to match the natural color of the patient's teeth. The liquid has 75 shades. 75 shades are available in 15ml, 30ml, 40ml, 50ml and 100ml volumes.

**C. Direction for use**

- (1) Milled zirconia teeth with framework must be kept dry before working.
  - (2) Remove any powders on milled zirconia teeth completely.
  - (3) Select suitable color of matching the color of natural teeth.
  - (4) Check the color of coloring liquid before use.
  - (5) Shake the coloring liquid sufficiently before opening the cap.
  - (6) Brush the coloring liquid by using brush.
  - (7) Brush the coloring liquid from one to five strokes to match tooth color.
  - (8) Sinter the zirconia framework in the furnace.
- Dipping**
- (1) Dip a milled zirconia into a coloring liquid within 30 seconds.
  - (2) Remove stagnant liquid with a tissue.
  - (3) Dry the dipped zirconia in a drying machine for 30 minutes at a temperature of 80°C or 176°F.
  - (4) In case of long bridge, dry the dipped zirconia for 60 minutes.
  - (5) Sinter the dried zirconia in the furnace. (sintering condition: refer to sintering condition of each block)

**Brushing**

- (1) Brush 1/3 of the cervical region with a suitable coloring liquid 3 times horizontally.
- (2) Brush 1/3 of the central area with one level lower coloring liquid 3 times vertically.
- (3) Brush 1/3 of the incisal region with two level lower coloring liquid 3 times vertically.
- (4) Brush the occlusal surface with two level lower coloring liquid 3 times.
- (5) Dry at room temperature for 5 minutes.
- (6) Brush it with effect liquid 2-3 times.
- (7) Dry the dipped zirconia in a drying machine for 30 minutes at a temperature of 80°C or 176°F.
- (8) In case of long bridge, dry the dipped zirconia for 60 minutes.
- (9) Sinter the dried zirconia in the furnace. (sintering condition: refer to sintering condition of each block)

**D. Cautions**

- (1) Use the coloring liquid with proper room ventilation always.
- (2) Avoid keeping the coloring liquid in temperatures more than 30°C or 86°F which might cause excessive evaporation of liquid and damage the container due to heavy internal pressure.
- (3) Keep the bottle of it upright always with right cap to seal the container completely.
- (4) Deterioration of the coloring liquid might cause negative effects on frameworks as in the following:
  - Discoloration, Distortion of shape during or after sintering, Reduced durability.
- (5) Always wear protective gloves and goggles to avoid direct contact of the coloring liquid with your skin or eyes which might cause irritation and harmful effect. Refer to safety advices in the below.
  - Eye contact : lift eye lid and flush your eye with clean running water sufficiently, more than several minutes. Seek medical advice.
  - Skin contact : Rinse with plenty of soapy water. Seek medical advice if necessary.
  - Ingestion : Do not induce vomiting nor drink water. Go to hospital immediately.
  - Inhalation : Move to other place and get fresh air. Seek medical advice. Bring the MSDS for consulting with doctors.
- (6) Ensure proper air ventilation during drying time of the coloring liquid.

**E. Storage**

- (1) Store at 1~30°C. If it is frozen in an environment under the temperature, just unfreeze it before use. Make sure bottles are not damaged due to freezing.
- (2) Do not use the coloring liquid beyond the validity period.

**F. Transportation, Maintenance after use**

- (1) Transport with cushioning material to buffer the impact.
- (2) Do not drop or throw.
- (3) The product is disposable and must not be reused.

**G. Warning**

- (1) Do not use this for other use.

**H. Expiration**

- (1) The expiry date is described on the container.

**[Pictograms]**

|  |                                  |  |                         |
|--|----------------------------------|--|-------------------------|
|  | Manufacturer                     |  | Keep dry                |
|  | Do not reuse                     |  | Caution                 |
|  | Date of Manufacture              |  | Use by                  |
|  | Batch Code                       |  | Catalogue Number        |
|  | Consult Instructions for Use     |  | Non Sterile             |
|  | Do not use If package is damaged |  | Keep away from sunlight |

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